



NaDCC

10gr x 100 tablets

Disinfecting Tablets Using Areas and Application

Recommended Klortab Strength	Water (litre)	Tablet Quantity (piece)	Disinfection Area	Application
10.000ppm	3	10	Body Fluid Spills and Blood and Serum Waste – against HIV (AIDS) and HBV (Hepatitis B Virus)- in 10% plasma, within 1 minute ultimate result.	Spills and Waste should be washed by the solution. Glove using.
5.000ppm	3	5	Disinfection of the areas which contaminated of body fluid spills may have HIV (AIDS) and HBV (Hepatitis B Virus) include operation rooms, morgue and autopsy rooms, laboratories. Stainless Steel Surgical Tools, urinal, bed pans, plastic materials, anesthesia items, in 10% plasma, within 2 minute ultimate result.	The surfaces washed by the Klortab solution. The stainless steel material will not be left in the solution more than 30 minutes. It is not recommended to use with covered metals. Use gloves. If necessary an anionic detergent can be added to the solution and at the same time cleaning and disinfection can be achieved.
1.000ppm	3	1	Operating rooms, morgue and autopsy rooms, Corridors, Burning Units, Intensive Care Units, Clinic and Pathology Laboratories	The surfaces washed by the Klortab solution. The clean materials left in the solution for 30 minutes. If necessary an anionic detergent can be added to the solution and at the same time cleaning and disinfection can be achieved.
200ppm	22,5	1	Kitchen surfaces, food stores, food preparation benches, wash basin, bath tubes, examination rooms, sterile services, corridors, waiting rooms, patient relationship desks, patient contact departments, around of hydrotherapy pools, toilets, stretchers...	The surfaces washed by the Klortab solution. The clean materials left in the solution for 30 minutes. If necessary an anionic detergent can be added to the solution and at the same time cleaning and disinfection can be achieved.
140ppm	30	1	Baby feeding bottles, pacifiers, feeding bottles covers, other feeding materials and toys, dirty and infected cloths, aprons, sheets etc cleaning cloths, mats, cloths using in the kitchen.	After cleaned the milk residues, the materials will leave for 30 minutes in the solution.

WARNING: THE DIRTY SOLUTIONS CAN BE POURED TO THE TOILETS FOR SEND AWAY THE INSECTS. KLORTAB NADCC DISINFECTING TABLETS KEEP THEIR ACTIVITY FOR 24 HOURS. THE TABLET SOLUTIONS OF KNOWN, ACCURATE STRENGTHS EFFECTIVE AGAINST TO BACTERIA, FUNGI, MYCOBACTERIA, SPORES AND VIRUSES (INCL. HIV) WITHIN 2 MINUTES AT 5.000PPM AND 1 MINUTE AT 10.000PPM

ALWAYS:

- Wear gloves,
- Use Clean and dry containers for the Klortab NaDCC Solutions
- Take care to measure out the Klortab NaDCC correctly.
- Make a fresh solution
- Wash away dirt (where appropriate) before disinfecting.
- Throw away solution at end of day or after use
- Ensure that tablets are completely dissolved before closing a container lid.
- In case of Poisoning Event, call free the phone of
- Poison Information Center at 114

AVOID:

- * Use with animal fibres such as silk or wool.
- * Prolonged immersion with stainless steel items.
- * Mixing acids and strong reducing agents.

Precautions: Store in cool and dry place. Do not swallow. Keep out of reach of the children. In case of contact with eyes, wash with abundant water and consult a doctor.

The recommended lifetime of the Klortab NaDCC solutions (When a visual pollution occurs, regardless of the solution should be changed immediately)

**1 – 1.000ppm = 1 day
1.000 – 2.500ppm = 2 days,
2.500 – 10.000ppm = 7 days**

MANUFACTURER:

ADA AQUA KİMYA SANAYİ VE TİCARET AŞ

**E.MAIL: info@adaaqua.com.tr
www.adaaqua.com.tr**

AUTHORIZED DISTRIBUTOR

ADA AQUA CHEMICAL INDUSTRY AND TRADE INC.

Çamlıca Mahallesi 147. Cadde Dimas İşyerleri Sitesi No:4-C Yenimahalle Ankara

**PHONE: 0312 387 00 47 FAX: 0312 387 00 43 GSM: 0554 870 50 01 E.MAIL: info@adaaqua.com.tr
www.adaaqua.com.tr**